
## **R**eplace Battery Holder


## INNOSONIAN



A. Prepare Brayden for replacement

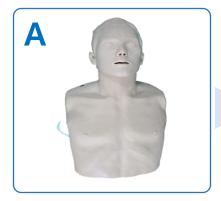

B. After taking off the body skin, detach rib frame

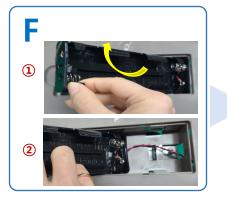

F. Lift the battery holder and remove the cable connected from the manikin to the battery holder




G. In reverse order of "F", place the cable of the new battery holder into the battery holder pocket on the body frame, then insert the battery holder well




C. After removing the screws in the numbered order, open main board cover




H. To fix the battery holder to the body frame, insert a screw into the center of battery holder

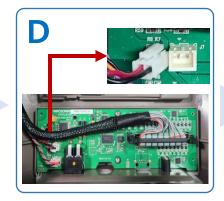


D. Detach battery holder cable's connector from main board

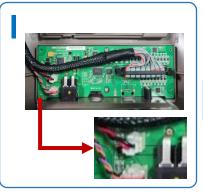

|. Attach battery holder cable's connector to the main board connector

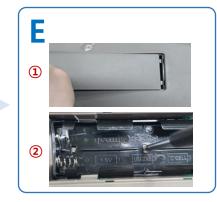

E. After removing the battery cover at the back of the manikin, unscrew the battery holder

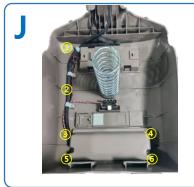

J. After organizing the cable as in the beginning, fix the line with 1st and 2nd clamps. Then close the main board cover and fasten the screws. Finally, after assembling rib frame as in the beginning, put body skin back