

CLEANING & DISINFECTING RECOMMENDATIONS

To ensure proper hygiene, we recommend a thorough cleaning of Baby Tyler after each use as per instructions below:



The manikin must be cleaned by disinfecting the lips and the face, removing the face skin from the head after each training session.

*Do not use bleach or ammonia-based products.



For more thorough disinfection, immerse the face skin in a container with water or disinfectant, wash carefully and keep dry for proper functioning.



After each use, the Baby Tyler mouthpiece must be disinfected and allowed to dry for correct functioning.



The Baby Tyler lung bag is semi-disposable and can be used to train group of people in a session without needing to change it.

TROUBLESHOOTING TIP


If the chest stops rising during a practice session, first ensure the mouth and airway system is properly connected to the lung bag and the head is positioned correctly. Check the lung bag for moisture. If the lung bag is moist, replace it with a new one.

ADD-ONS & REPLACEMENT PARTS

#11-220	Baby Tyler® Lung Bags - 10 Pack
#11-225	Baby Tyler® Mouthpiece - Light Skin - 10 Pack
#11-245	Baby Tyler® Mouthpiece - Dark Skin - 10 Pack
#11-226	Baby Tyler® Face Skin - Light Skin
#11-233	Baby Tyler® Face Skin - Dark Skin
#11-227	Baby Tyler® Chest Skin - Light Skin
#11-235	Baby Tyler® Chest Skin - Dark Skin
#11-247	Baby Tyler® Feedback Monitor
#11-248	Baby Tyler® Clothing



FOR MORE HELPFUL TIPS & TRICKS, SCAN HERE!

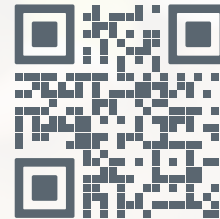

Federal Communications Commission Statement and Industry Canada Statements
BABY TYLER - MANIKIN FCC ID: 2A2SDCPR1Y1 IC: 27592-CPRTY1 HVIN: VM-BABY MANNEQUIN
BABY TYLER - MONITOR FCC ID: 2A2SDCPR2Y2 IC: 27592-CPRTY2 HVIN: VM-BABY MONITOR

Changes or modifications not expressly approved by Vicedo Martí S.L. could void the user's authority to operate the equipment.

This device complies with Part 15 of FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions: 1. This device may not cause interference. 2. This device must accept any interference, including interference that may cause undesired operation of the device.

L'émetteur/récepteur exempt de licence contenu dans le présent appareil est conforme aux CNR d'Innovation, Sciences et Développement économique Canada applicables aux appareils radio exempts de licence.

L'exploitation est autorisée aux deux conditions suivantes : 1. L'appareil ne doit pas produire de brouillage; 2. L'appareil doit accepter tout brouillage radioélectrique subi, même si le brouillage est susceptible d'en compromettre le fonctionnement.



WORLDPOINT ECC, Inc.
 1326 South Wolf Road, Wheeling, Illinois 60090, USA
 Phone: +1 (847) 465-3200
worldpointproducts.com



USER MANUAL

INCLUDES

- 1 infant manikin
- 1 wireless monitor
- 4 lung bags
- 2 mouthpieces
- 2 9V batteries
- 1 transport bag



REALISTIC BREATHING

The head tilt enables students to learn the correct breathing technique



OROPHARYNGEAL AIRWAY

To practice with a oropharyngeal airway, remove the mouthpiece, replace the skin, measure and insert the airway



AED TRAINING

Pads do not leave residue



CHOKING PRACTICE

Train infant choking relief skills including back slaps

3-year warranty



WIRELESS MONITOR



ELECTRONIC FEEDBACK



BLIND MODE



TEST REPORT

ASSEMBLY



1. Remove the chest skin
2. Remove the face skin
3. Move the lung bag below the jaw
4. Connect the lung bag to the mouthpiece



5. Place the lung bag on the chest, and make sure the lung bag lines up with the bottom line of the chest square
6. Make sure the lung bag is attached to the chest using the sticker

7. Fit the face skin, checking that the nose, mouth & upper tab are in the correct position



8. Place chest skin and attach to the studs

KEY FEATURES & FUNCTIONS

SETTING UP BABY TYLER®

Insert one 9V battery into the torso and one into the monitor (included).



Press the power button on the lower left corner of the monitor to turn on; lights will illuminate. **Press down hard & all the way** on the manikin's chest to turn it on. A beep indicates that the torso has paired with its monitor; confirm by checking that the light next to "Torso" is illuminated green.

COMPRESSION FEEDBACK

RATE
100 ↔ 120 bpm



DEPTH & RECOIL

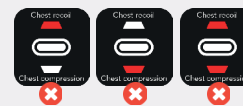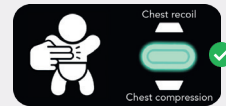

Wait 10 seconds
for test report

STANDARD MODE



BLIND MODE





Feedback is not visible during practice in blind mode. View the test report 10 seconds after the session has finished in both modes.

MONITOR RESET

To reset, turn the monitor on, then press and hold the "Mode" button for 7 seconds until the rate & compression icons flash.

Connect the torso by press down on the manikin torso as deep as it will go and hold until you hear a beep to confirm pairing.

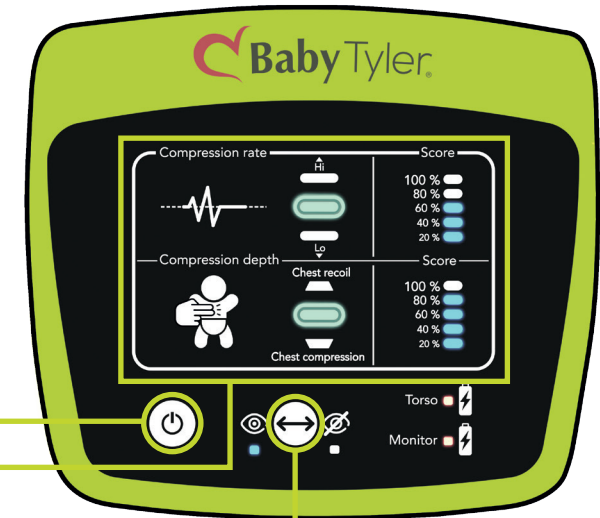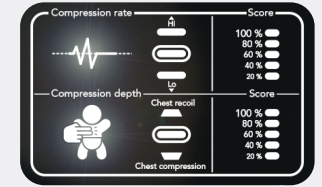

BATTERY CAUTION

The batteries supplied with Baby Tyler should be replaced when the red light appears on the monitor indicator.

Using uncharged batteries with either the monitor or the torso will impede proper functioning of Baby Tyler.

There are no interference implications using rechargeable batteries.

