

# Meet Baby Tyler®

## THE LIFELIKE CPR MANIKIN REPRESENTATIVE OF REAL INFANTS

- Realistic size & movements
- 2020 AHA guideline compliant CPR feedback



### ASSEMBLY



1. Remove the chest skin
2. Remove the face skin
3. Move the lung bag below the jaw
4. Connect the lung bag to the mouthpiece



5. Place the lung bag on the chest, and make sure the lung bag lines up with the bottom line of the chest square
6. Make sure the lung bag is attached to the chest using the sticker



7. Fit the face skin, checking that the nose, mouth & upper tab are in the correct position



8. Place chest skin and attach to the studs


### TROUBLESHOOTING TIP



Baby Tyler®'s fully functional nose requires that the face skin (and specifically the nostril skin) is properly lined up. Failure to properly line up the nostril skin within the nose piece (and the mouth skin with the mouthpiece) will result in improper airway functioning.

If the chest stops rising during a practice session, check that the lung bag is not moist. If the lung bag is moist, replace it with a new one.

### MONITOR GUIDE

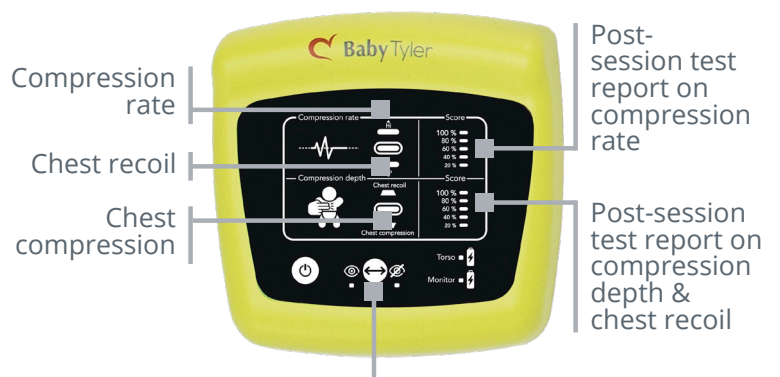

Standard & Blind Modes

View the test report 10 seconds after the session has finished in both modes